

Handicare 2000 Fits Your Life

- The Handicare 2000 is designed for stairs with curves, corners or a spiral.
- It's twin rail track comes in two standard colors, but can be special ordered to match any decor.
- Choose from six attractive upholstery colors to match the stairlift to your home's interior. The upholstery wipes clean easily and is flame-retardant.
- Adjust the seat to match your height for a comfortable ride.
- The rack and pinon drive ensures a smooth and quiet ride.
- A certified engineer installs your stairlift quickly and cleanly. In most homes, the work only takes a few hours and you can use your lift the same day.
- The Handicare 2000 is battery powered so you will never be stranded during a power outage.
- If you have difficulty bending at the hip or knee, choose the optional perch seat.
- Overcome the challenge of outdoor staircases with the Handicare 2000 Outdoor™. This UV-protected, weatherproof stairlift has a waterproof cover so it is always dry and ready.
- The Handicare 2000 is energy efficient – it only uses about \$7 of power in a year.

Safety Standard

The Handicare 2000 has safety sensors on the power pack and footplate. These sensors are designed to stop the lift if it meets any obstruction. Once stopped it can only move in the opposite direction to the obstruction to allow for removal.

In addition, all Handicare stairlifts have a diagnostic display so that you can monitor the status of your stairlift, for example if a safety edge has been activated.



Safety Standards

The easy release retractable seat belt is a standard safety feature that can be fastened on either side of the seat, or centrally, whichever is easier for the user.

The Handicare 2000 also has an easily accessible key switch that allows you to disable the lift and prevent unauthorized use. So you don't need to worry about unsupervised children playing with your stairlift.



Easy to Use

The Handicare 2000 is operated by pressing or holding the large toggle switch in the direction of travel.

Located at the end of the armrest, the switch can be fitted on either arm and is easy to use, even for those with conditions such as arthritis.



The footplate finishes level with the landing making it easy to get on and off your stairlift.



Handicare 2000 at a Glance

Seat options		Comments
Choice of Upholstery	✔	A selection of six colors means you can choose upholstery to match your decor.
Intelligent Safety	○	Automatically returns your stairlift to the safest park position to facilitate emergency access.
Powered Swivel	○	This makes turning the stairlift seat to get in and out on the landing as easy as possible.
Retractable Seat Belt	✔	The retractable cream belt can be clipped together on either side or in the center.
Velcro Seat Belt	○	The black belt has velcro on either end to hold it firmly in place.
Perch Seat	○	Ideal for users who have difficulty bending at the hip or knee joint.

Footplate options		Comments
Manual	✔	The footplate is folded and unfolded manually.
Powered from Seat	○	Automatically folds the footplate away when the seat is folded up.
Powered from Arm	○	The footplate can be lifted and lowered by a switch under the arm while the user is seated.

Track features		Comments
Travels Around Corners and Bends	✔	Designed for a wide range of staircases: from ones that turn a corner to multi-flight or spiral.
Multiple Charge Points	✔	Use your remote control to park on charge points.
Track Available in Any Color	○	Customize your curved stairlift track to match your decor.
Zero Intrusion onto Landing	○	For use when a doorway close to the top of the stairs would be obstructed by the track.
Powered Hinge	○	Automatically moves the track while the stairlift moves leaving doorways unobstructed.

Weight Capacity	
Standard Maximum Weight	254 lbs
Max Weight with Heavy Duty Kit	302 lbs

Standard Compliance	
BS5776: Stairlifts	✔
ISO9386: Stairlifts	✔
CSAB613, CSAB355 & CSAB44.1	✔

✔ Available as standard ✖ Not available ○ Optional



Handicare * 2201 Hangar Place, Suite 200 * Allentown, PA * Tel: 610 266 5260 * Fax: 610 266 5266 * Toll Free: 866 276 LIFT/5438 *
www.handicare.com * customerservice.usa@handicare.com

2000
Curved Stairlift



Keep Your Independence
Stay in the Home You Love



Are Your Stairs a Challenge?

Stay Independent with a Stairlift from Handicare

Do you or someone you love have difficulty going up and down the stairs? Are you concerned about safety?

You are not alone. Every year, thousands of people look for a solution to the problems posed by the difficulty they have using the staircase in their home.

You don't have to be at risk. The decision to install a stairlift is sometimes a difficult one, but when compared to the alternatives of moving or limiting access to one level of your home, it is many times the preferred solution.

You don't have to worry. A Handicare stairlift offers you a safe and reliable solution. Our team of experts have assisted thousands of people in overcoming problems posed by going up and down their stairs. We have many models of lifts to accommodate your needs, and a certified engineer will install your stairlift usually in a couple of hours. So you will have the safety and comfort you need to keep your independence and live in your home free from worry.




"I was concerned that having a stairlift in our home was going to look to industrial, but I was wrong. It fits the form of our staircase and the look of our home. Plus, the independence I maintained is a benefit that is priceless."

Choosing a Stairlift from Handicare

Unless you know someone who is already benefitting from using a stairlift, you may be a little unsure of what is involved in owning and using one.

Every stairlift has three basic components. The track is fitted to your staircase, not the wall as is commonly thought, and a power pack (the drive unit) is fitted to the track. Then the seat that most fits your needs is fitted to the power pack.

Stairlift tracks fall into two basic categories; straight and curved. Straight stairlifts are suitable for use where a stairlift only needs to travel in one straight line and curved stairlifts have tracks that are specifically designed to suit staircases that turn corners or are spiral.

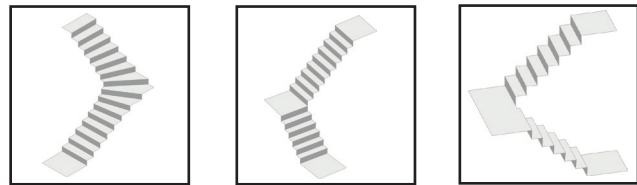

The twin rail system, designed for a stable and smooth ride, comes in light grey or gun metal as standard, or can be special ordered in any color to suit your decor.

All curved tracks are made to order following a survey of each individual staircase.

The **Handicare 2000** suits most staircases with bends or corners, and offers you all the standard features of a Handicare stairlift as well as powered options.

Swiveling Your Stairlift

When you reach the top of the stairs you will need to swivel your seat around so that you exit onto the landing safely.



Manual swivel

If you have the manual swivel you will find a handle at either side of the seat. By pressing either handle down, or pulling it up, you can swivel the seat around to face the landing, where it will lock in place, allowing you to step off.

Powered Swivel

If you choose the powered swivel option the swivel is operated by continuing to hold the toggle when you arrive at the top of the stairs. The seat will automatically move while the toggle is activated. Our powered swivels have a manual override for emergency release to improve stairway access if required.



Folding Your Footplate

The footplate and seat can be folded up to increase available space leaving the staircase as clear as possible for other users.

Manual Footplate

Lifted and lowered by hand, manual footplates are fitted with an easy fold mechanism that minimizes the strength required to fold the footplate up or down.

Powered Footplate

If you choose the powered footplate option it will remove the need for you to bend down to fold or unfold the footplate. You can choose to have a footplate that folds automatically when the seat is folded and unfolded, or that is operated separately to the seat by a switch in the seat arm so you can raise and lower the footplate while seated.



Parking and Calling Your Stairlift

The Handicare 2000 comes with two remote controls so you can have one upstairs and one downstairs. This means you can send your stairlift up and down the track without having to sit on it.

Use your remote control to park your stairlift on the charging points at the top or bottom of the stairs. Plus it is ideal for multi-user situations where a user can call the stairlifts between levels, just like an elevator.



Hinged Track

In order for the seat to get low enough for the user to sit, it is necessary for the track to extend beyond the staircase to the floor. Sometimes a doorway may be close to the bottom of the stairs and the track may intrude across it. In these circumstances a hinged track should be fitted. This means that the track can be folded away from the doorway giving clear access.

Powered Hinge

As an upgrade, the Handicare 2000 is available with an automatic hinged track. When operating the stairlift from the toggle, the hinge will lift/lower automatically.

When the stairlift is not in use the hinge is folded by holding a button on the remote control which means you do not need to move the track yourself. When the button is pressed your stairlift moves up the track and parks; the hinge section then moves automatically leaving the doorway clear.



Additional Seat



Perch Seats

If you have difficulty sitting or have a tight staircase, a perch seat may be the solution.

With a perch seat you are in an almost standing position when you are using the lift. There is a shortened seat and retractable seat belt which provide a stable support area for users to "perch" on.

Upholstery is available in your choice of six colors.